Essix Tray Rite® Plastic

(Instructions for use on Essix® Vacuum Machines Only)

Indications for Use:

Essix Tray Rite® Plastic is thermoformed to fabricate intra-oral tooth whitening (bleaching) trays.

Contraindications: None known.

Warnings:

Excessive heating time during thermoforming will cause plastic to over sag and potential for injury to operator.

Improper thermoforming of the plastic appliance may cause the appliance to crack/break, resulting in sharp edges, loose pieces and possible aspiration of pieces.

Precautions:

Patients with a history of allergic reactions to plastics should not use this product.

Store plastic sheets in cool, dry place.

Single Use Only.

Adverse Reactions: None known.

Directions for Use:

1. All models should be trimmed to ¾" high. Full arch models should have the palate trimmed away.
2. **IMPORTANT:** Pre-heat the machine before using! Pre-heat the vacuum machine for at least 3 minutes or until the heating element is a solid orange color. (Heating times may vary with current fluctuations or length of time the heating element has been left on.)
3. Spray the model with Trim Rite® Release Agent to prevent tray material from sticking to the model.
4. Place the plastic into the frame on the vacuum machine and the model on the base plate.
5. Swing the heating element over the plastic. Observe the plastic as it heats.
6. Essix Tray Rite® Plastic will become wavy then flatten to its original position – this is the optimal thermoforming time.
7. Move the heating element away from the plastic.
8. Turn on the vacuum and lower the plastic over the model.
9. Immediately spray the plastic encased model with Essix Freeze Spray® Coolant. This is an essential step. Not only does it cool the plastic prohibiting further thinning, but it “shrinks” the material around the cast, realizing a tighter fit.

10. Let vacuum continue to run until plastic is cool.

11. Trim and finish appliance as desired.

Heating time may vary depending upon machine. If plastic does not adapt to model, add heating time until adaptation occurs. If plastic “folds” when thermoformed, reduce heating time until material does not fold.

Thickness	Vacuum Heat Time*	Biostar™ Code*	Drufomat™
0.5mm (.020")	25 seconds	113	55 seconds

*after preheating element

Manufactured by:

DENTSPLY

Raintree Essix Glenroe

1912 44th Avenue East

Bradenton, FL 34203

Tel: 800.883.8733

www.essix.com

Emergo Europe

Molenstraat 15,

2513 BH, The Hague

The Netherlands

Tel: (+31) 70 345 8570

Fax: (+31) 70 346 7299

Biostar and Drufomat are not trademarks of DENTSPLY. Essix TR DFU - ENG #MKTR RTE-042-10 Rev. 03/10

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