


Raintree Essix, Inc.
Ortho Box® Storage System

CONTENTS

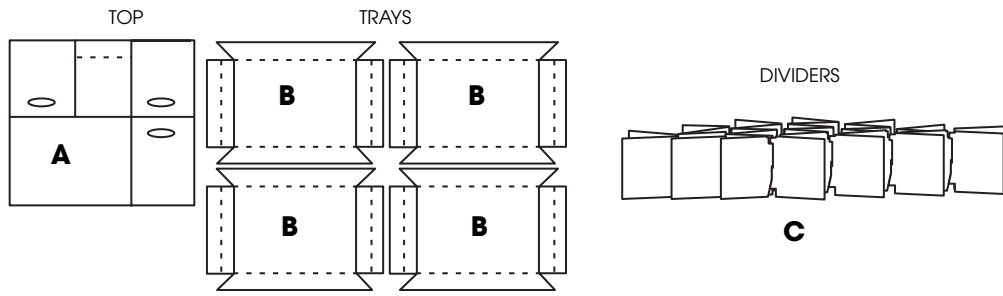

A – Box (1) **B** – Trays (4) **C** – Dividers

CONSTRUCTING THE BOX

STEP 1

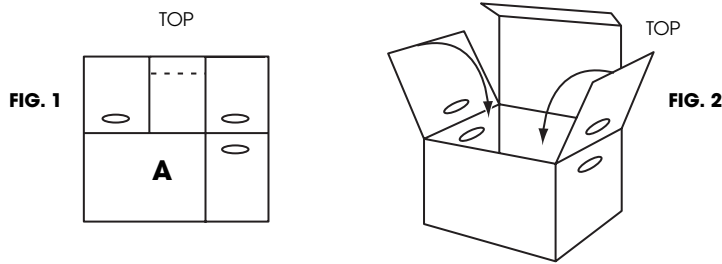


Figure 1: Select piece A and hold vertically.

Figure 2: Form box by separating the front from the back. Bottom will lock into place. Fold flaps in.

FORMING THE TRAY

STEP 2

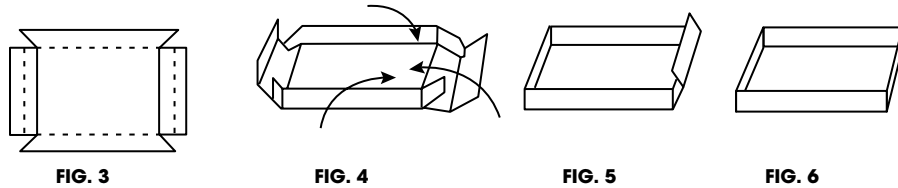


Figure 3: Select 1 of 4 trays (piece **B**).

Figure 4: 1) Begin to form tray by folding long side of tray. 2) Fold short flaps at each end of tray.

Figure 5: Form end of tray by folding notched flaps over sort flaps. Insert in notches into base.

Figure 6: Repeat steps in Figure 5 at other end.

PLACING DIVIDERS INTO THE TRAY

STEP 3

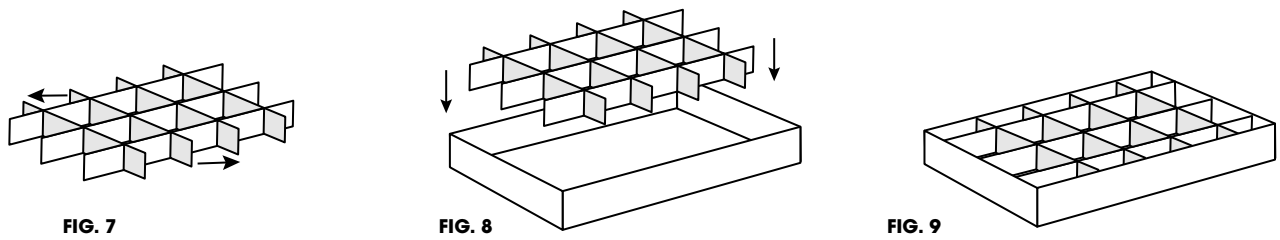


Figure 7: Pull ends of dividers (**C**) in opposite directions until grid opens up.

Figure 8: Insert dividers into tray.

Figure 9: Complete tray should be divided into 20 compartments.

INSERT TRAYS INTO ORTHO BOX®

STEP 4

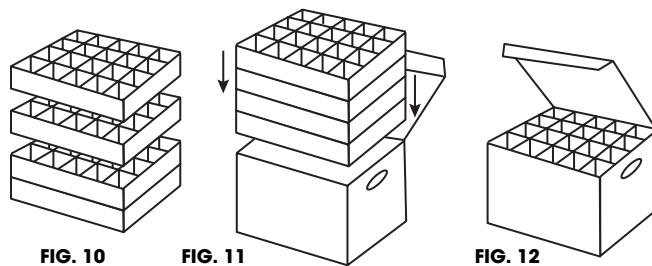


Figure 10: Repeat the steps for forming trays and placing dividers for each of the 4 trays and stack.

Figure 11: Insert trays right side up into Ortho Box®.

Figure 12: Completed Ortho Box® holds 4 trays of 20 compartments.

RAINTREE ESSIX®
 INC.